

SHOFU DENTAL ABRASIVE KITS

Step by Step Instructions

Composite Polishing Kit CA

Item No. 0310B

This slow speed kit is for finishing and polishing all types of composites.

- Step 1:** Use Dura-White Stones for contouring, finishing and minor adjustments.
- Step 2:** Use CompoSite Regular Polishers (no band) for finishing and pre-polishing.
- Step 3:** Use CompoSite Fine Polishers (white band) for final polish.



Porcelain & Enamel Polishing Kit CA

Item No. 0317B

This slow speed kit is for finishing and polishing all porcelains and enamel.

- Step 1:** Use Dura-White Stones for minor adjustments and finishing.
- Step 2:** Use Ceramiste Standard Polishers (no band) for pre-polishing.
- Step 3:** Use Ceramiste Ultra Polishers (yellow band) for fine polishing.
- Step 4:** Use Ceramiste Ultra II Polishers (white band) for high luster final polish.



Amalgam & Metals Polishing Kit FG

Item No. 0309B

This high-speed kit is for finishing, polishing and super polishing amalgams, precious and non-precious alloys.

- Step 1:** Use Brownie Polishers for pre-polishing.
- Step 2:** Use Greenie Polishers for polishing.
- Step 3:** Use Super-Greenie Polishers (yellow band) for high luster final polish.




Amalgam & Metals Polishing Kit CA

Item No. 0308B

This slow speed kit is for finishing, polishing and super polishing amalgams, precious and non-precious alloys.

- Step 1:** Use Brownie Polishers for pre-polishing.
- Step 2:** Use Greenie Polishers for final polish.

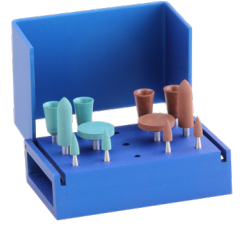

Acrylic Polishing Kit HP

Item No. 0321B

This slow speed kit is for adjusting and polishing acrylics.

- Step 1:** Use Robot Carbide HP Cutter to make major adjustments on acrylics.
- Step 2:** Use AcryPoint Coarse Polishers (dark grey) to adjust.
- Step 3:** Use AcryPoint Medium Polishers (brown) to finish.
- Step 4:** Use AcryPoint Fine Polishers (light grey) for high luster final polish.




Zirconia Polishing Kit CA

Item No. 0333B

This slow speed kit is used on Zirconia, Lithium Disilicate and Glass Ceramic restorations. This kit contains diamond impregnated polishers that will finish and polish.

- Step 1:** Use ZilMaster Coarse Polishers (green band) for pre-polishing.
- Step 2:** Use ZilMaster Medium Polishers (blue band) for polishing.
- Step 3:** Use ZilMaster Fine Polishers (yellow band) for high luster final polish.

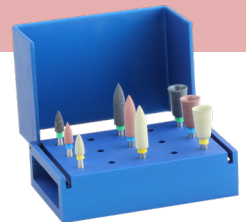



Composite Polishing Kit CA (\$79.95)
(Item No. 0310B)

- Four CompoSite Midi-Pts., two of each Regular & Fine
- Four CompoSite Cups, two of each Regular & Fine
- Two CompoSite Knives, one of each Regular & Fine
- One Dura-White® FL2
- One Dura-White® RD2




Porcelain & Enamel Polishing Kit CA (\$79.95)
(Item No. 0317B)

- Five Ceramisté® Midi-Pts., two of each Standard, Ultra & one Ultra II
- Three Ceramisté® Cups, one of each Standard, Ultra & one Ultra II
- Two Ceramisté® Wheels, one of each Standard & Ultra
- One Dura-White® FL2
- One Dura-White® RD2



Amalgam & Metals Polishing Kit FG (\$79.95)
(Item No. 0309B)

- Seven Mini-Pts., three of each Brownie® & Greenie® & one Supergreenie
- Five Cups, two of each Brownie® & Greenie® & one Supergreenie

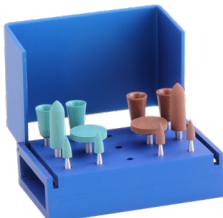


Amalgam & Metals Polishing Kit CA (\$79.95)
(Item No. 0308B)

- Four Mini-Pts., two of each Brownie® & Greenie®
- Four Cups, two of each Brownie® & Greenie®
- Two PC2's, one of each Brownie® & Greenie®
- Two WH6's, one of each Brownie® & Greenie®



Acrylic Polishing Kit HP (\$134.95)
(Item No. 0321B)

- Three AcryPoint BP1's, one of each Coarse, Medium & Fine
- Three AcryPoint PC's, two of each Coarse, Medium & Fine
- One Robot® Carbide HP Cutter SH 79E
- One Robot® Carbide HP Cutter SH 251E

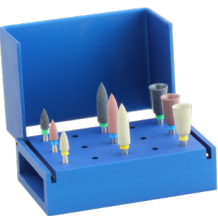

Zirconia Polishing Kit CA (\$179.95)
(Item No. 0333B)

- Three ZiLMaster Bullets, Coarse, Medium & Fine
- Three ZiLMaster Cups, Coarse, Medium & Fine
- Three ZiLMaster Mini-Pts., Coarse, Medium & Fine

